

 t:
 01244 287050

 f:
 01244 289175

 e:
 sales@venalink.co.uk

 www.venalink.co.uk

 Unit 46, First Av, Zone 2, Deeside Ind. Park CH5 2NU

## HOW TO SEAL - DAILY





## 1. ALIGN

- Crease the card along the fold
- Place your daily card on the template
- Place in the blister



## 2. PEEL & FOLD

- Remove the backing paper and fold over the foil panel.
- Smooth over the card with your hand.



## 3. ROLL

• Using your roller, roll the card ensuring enough pressure is applied.