t: 01244 287050

f: 01244 289175

www.venalink.co.uk

e: sales@venalink.co.uk

Unit 46, First Av, Zone 2, Deeside Ind. Park CH5 2NU

HOW TO SEAL - MONTHLY BIFOLD YELLOW





1. **ALIGN**

- Crease the card along the fold
- Place your monthly card on the template
- Place in the blister



2. **PEEL & FOLD**

- Remove the backing paper and fold over the foil panel.
- Smooth over the card with your hand.



3. **ROLL**

Using your roller, roll the card ensuring enough pressure is applied.