

t: 01244 287050
f: 01244 289175
e: sales@venalink.co.uk
www.venalink.co.uk

Unit 46, First Av, Zone 2, Deeside Ind. Park CH5 2NU

HOW TO - USE YOUR PIL-BOB



1. **LOCATE** the blister for the correct day of the week and the correct time of day



2.TURN over the card



3. Remove the contents of the blister by **INSERTING** the serrated tip of the Pil-Bob into the foil and sweeping around the backing.



4. Allow the medication to fall into the Pil-Bob cup